

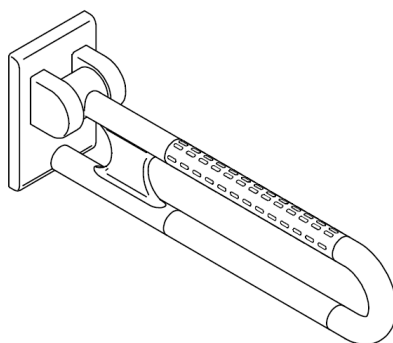
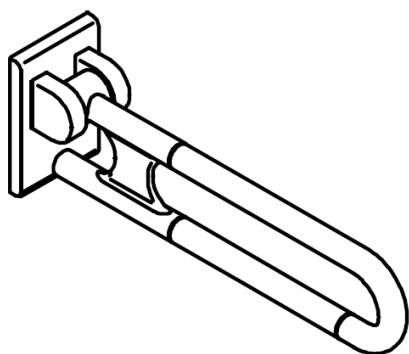
NORMBAU

PASSION FOR CARE

- (DE) Montage- und
Gebrauchsanleitung
- (EN) Fixing instruction and
directions for use
- (FR) Instructions de montage et
d'utilisation
- (NL) Montage- en
gebruiksaanwijzing

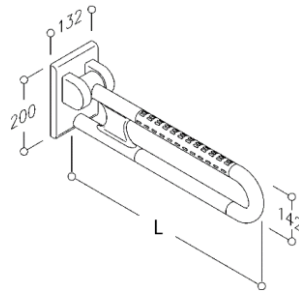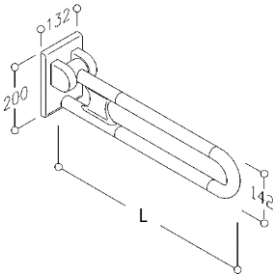
Nylon Care


EDV Nr.	L (mm)	Masse (kg)
0300 471	600	3,6
0300 472	725	4
0300 473	850	4,4
0300 474	900	4,6


EDV Nr.	L (mm)	Masse (kg)
0447 010	600	4
0447 030	725	4,5
0447 050	850	4,9
0447 070	900	5

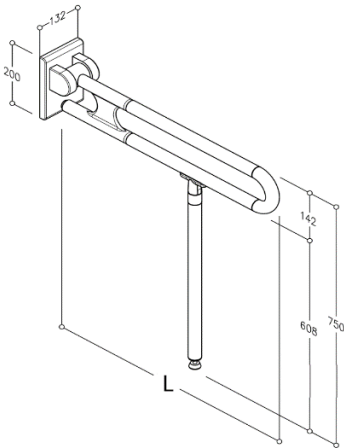
0300 455 ... 0300 458

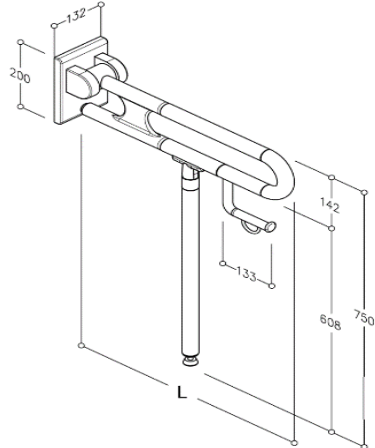
0300 455 ... 0300 458

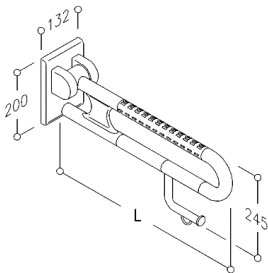
0447 120 ... 0447 180

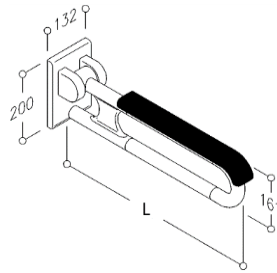
0447 410 ... 0447 470

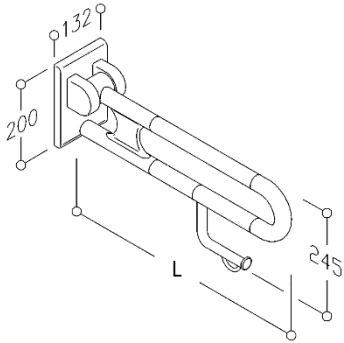0300 475 ... 0300 478

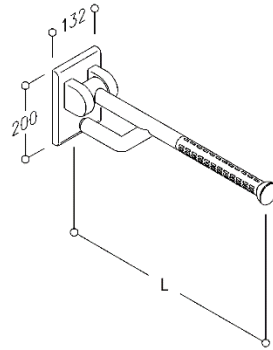0437 010 ... 0437 070

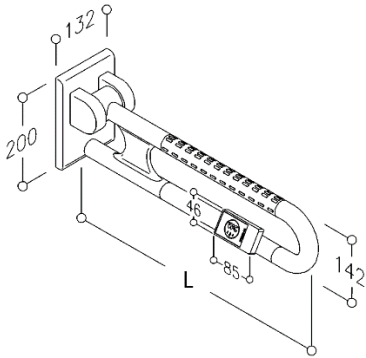0447 361 ... 0447 377

0300 479
Nylon 300 / 400

Belastung

Max. zul. Benutzergewicht: 0300 ... & 0437 ... : 100kg / 0447 ... : 125kg

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Load

Max. permissible user weight: 0300 ... & 0437 ... : 100kg / 0447 ... : 125kg

With max. length and max. permissible user weight, the max. force per fastening is 3kN.

If the max permissible user weight is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

Warning notices

Installation is to be carried out by a specialist and only with NORMBAU mounting sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material.

Observe the information on the max. permissible user weight and the instructions for mounting!

Care and cleaning instructions

Nylon products from NORMBAU have a smooth, high-gloss, easy-to-clean surface. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common disinfectants can be used for wipe disinfection. Acidic cleaning agents, care agents, disinfectants and decalcifying agents must always be used in accordance with the instructions for use. An acid content of over 10% can attack the surface and negatively change the brilliance and color. Use all agents only diluted, leave to act briefly and rinse off sufficiently with water. Abrasive, strongly acidic and alcoholic cleaning and care products should be avoided with Nylon.

Intended use

NORMBAU products are designed for use in wet rooms, at home, in care facilities and hospitals. They offer support to people with and without physical limitations. Use by children is not permitted.

Articles that are no longer required must be disposed of in the prescribed manner. Listed articles are made of recyclable polyamide and metal components.

Maintenance

NORMBAU products are not maintenance-free! The products are to be tested monthly by qualified specialists for proper function. In the event of defects (e.g. insufficient braking force), the assembly instructions must be observed.

Attention

There is no toxic hazard from the material used by NORMBAU. Tested according DIN EN ISO 10993-5:2009. Sun exposure or cleaning with hot water can reach temperatures higher than 41°C, which can lead to skin damage / burns.

Caution, risk of injury

When folding the support arm upwards, do not put your hand between the wall and the handle. When folding the handle down, do not hand between the buffer and the wall plate!

Disclaimer of liability

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions. According to the current regulation, the warranty is two years.

Under the intended conditions of use and in compliance with the guidelines for maintenance and cleaning, the intended lifespan of the article is five years.

If you would like more information about the products, visit us at www.normbau-extranet.de or contact us via customer service +49 (0) 7843 704 0.

For an enlarged view, you can find these assembly instructions by entering the article number on the homepage

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NORMBAU GmbH